

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed.

The appliance is only to be used with the provided power supply.

READ ALL INSTRUCTIONS BEFORE USING

DANGER - To reduce the risk of electrocution:

- Always unplug the SteriShoe® shoe sanitizer immediately after using.
- Do not use while bathing.
- Do not place or store the SteriShoe where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for a SteriShoe sanitizer that has fallen into water. Unplug immediately.

WARNING - To reduce the risk of burns, electrocution, fire, or injury to persons.

- The SteriShoe should never be left unattended when plugged in.
- Close supervision is necessary when the SteriShoe shoe sanitizer is being used by, on, or near children or invalids.
- Use the SteriShoe only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate the SteriShoe if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water.

- Keep the cord away from heated surfaces.
- Never drop or insert any objects into any opening.
- Do not look at the UV light bulb when it is on.
- Do not use the SteriShoe outside a shoe.
- Treat the SteriShoe carefully. You are placing a fragile light bulb into a shoe.


- If the SteriShoe does not turn off after approximately 30 minutes, the timer may not be functioning properly. Turn the unit off, disconnect the power, and please call 1-800-PEDIFIX (733-4349).
- If the controller light is flashing, inspect the UV light bulb to be sure that it is not broken. If the light bulb is broken, DO NOT put your foot into the shoe until you have removed all of the pieces of the broken light bulb.

- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

- Children should be supervised to ensure that they do not play with the appliance.
- To clean, disconnect Power and wipe with a soft dry cloth. Do not use alkaline cleaning agents.

SAVE THESE INSTRUCTIONS

Important Information

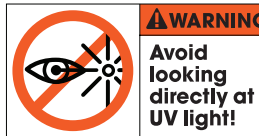

DO NOT look at the UV light when the SteriShoe bulbs are illuminated. UP-CLOSE, PROLONGED EXPOSURE to UV light can be harmful to

eyes and skin. A quick look, from 2-feet or further away, is harmless.

If lights on the controller are flashing, an error has occurred. The SteriShoe has built-in safety features to reduce the likelihood of accidental exposure to the UV light bulb. An error is most likely due to the activation of a safety feature. See the troubleshooting section for more information.

UV LIGHT BULB CONTAINS MERCURY (Hg). The UV light bulb contains a fine metal ball which you may hear inside the light bulb. This is normal; the light bulb is not broken. Since mercury is toxic please dispose of the light bulb in accordance with local disposal laws.

Specifications

SteriShoe ultraviolet shoe sanitizer,
Model: P3400, 12VDC, 1,2 A
Power Adaptor: Input: 100 - 240 V, 50/60 Hz, 0.4A Max., Class VI;
Output: 12VDC, 1.5A
Conforms to UL STD. 1431.
UV light bulb: 5W with a peak frequency of 253.7 nm.

Correct disposal of this product



This marking indicates that this product should not be disposed with other household waste. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources.

To return your used device, please use the return and collection systems or contact the retailer where the product was

purchased. They can take this product for environmental safe recycling. Refer to www.pedifix.com for more information.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio / TV technician for help.

Warranty Information

Limited One Year Warranty

Warranty coverage: PediFix, Inc. warrants, to the original purchaser, that this product will be free of defects in materials or workmanship for a period of one (1) year* from date of purchase, with the exceptions stated below.
*The warranty term for UV light bulb is 60 days from the date of purchase.

What is not covered: This warranty does not cover damage caused by accident, tampering, improper care, misuse, uses for which the product was not designed, or other causes not arising from defects in material or workmanship.

Warranty performance: During the above warranty period, PediFix will repair or replace any part of the product deemed defective. Contact PediFix at 1-800-PEDIFIX (733-4349) to obtain warranty service or go to www.pedifix.com for more information.


Warranty disclaimers: Any implied warranties arising out of this sale, including but not limited to the implied warranties of description, merchantability and fitness for a particular purpose, are limited in duration to the stated warranty coverage. In no event shall PediFix Inc. be liable for loss of use of this product or for any indirect, special, incidental or consequential damages, or costs, or expenses incurred by the consumer or any other user of this product, whether due to a breach of contract, negligence, strict liability in tort or otherwise. PediFix Inc. shall have no liability for any personal injury, property damage or any special, incidental, contingent or consequential damage of any kind resulting from the use of this product. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of consequential or incidental damages, so the above limitation may not apply to you.

Legal remedies: This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

Quick Trouble-Shooting Tips

1. The UVC bulbs in your SteriShoe may become loose during shipment. If a bulb does not illuminate, unplug the device, then gently remove, rotate and re-install it, and try again.

For assistance, call 1-800-733-4349.

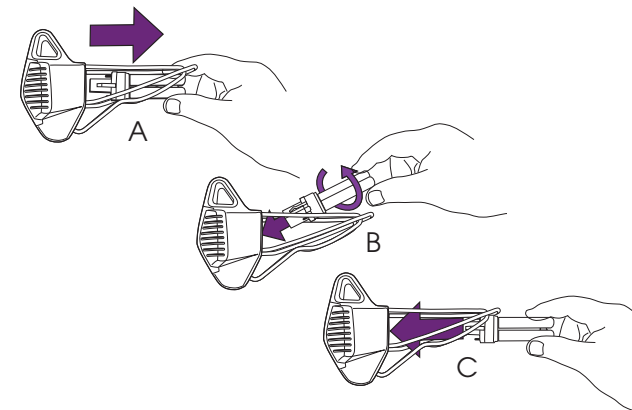

2. When in use, be sure the device — and its black Safety Sensor — sits flat inside your shoes. If not, as a safety precaution, it will not function.

Disclaimers

- The SteriShoe destroys up to 99.9% of the targeted microorganisms. Targeted microorganisms are those that come in direct unobstructed contact with the UV light.
- SteriShoe is not a medical device and does not treat a medical condition. It kills germs in your shoes that could come in contact with your feet.
- UV light may discolor or degrade some materials on the inside of the shoe.

© 2025 PediFix, Inc.
PediFix Footcare Company
301 Fields Lane
Brewster, NY 10509

SteriShoe #P3400
REV1224

SteriShoe®


UV Germ + Odor Eliminator
For Best Results, Use Daily

Instructions




www.pedifix.com
1-800-PEDIFIX (733-4349)

SteriShoe sanitizer parts

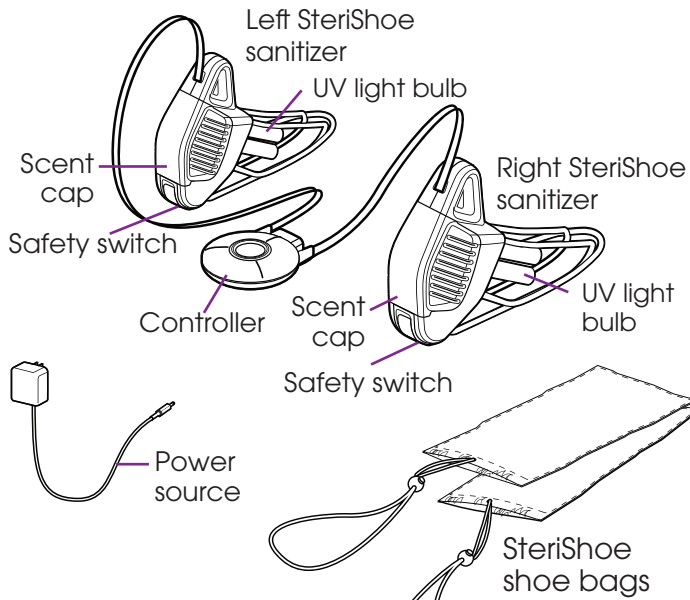

Controller buttons

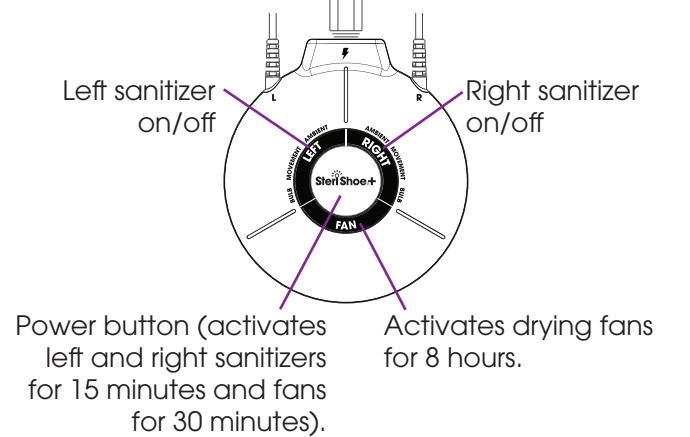

Operating SteriShoe+ sanitizers

1. Plug power source into the controller.
 2. Plug power source into standard wall outlet (100-240V).
 3. Insert right SteriShoe+ sanitizer into right shoe and left SteriShoe sanitizer into left shoe.
 4. Place each sanitizer and shoe into SteriShoe bag and tighten each shoe bag. (Required for open shoes such as sandals).
 5. Press SteriShoe button on the controller to activate both sanitizers.
- For damp or wet shoes:**
Press the fan button on the controller for 8 hours of drying time.

Inserting/replacing SteriShoe UVC Bulbs

1. Make sure the SteriShoe is unplugged.
2. Grasp top and bottom of the dual bulb component, gently pull straight out of socket.
3. To insert, gently push dual bulb component straight into socket.

Inserting/replacing scent pouch

Regular use of scent pouches will help deodorize and refresh your shoes.

1. Make sure the SteriShoe is unplugged. Use fingernail or coin to gently pry open the WHITE Scent Pouch Compartment cover. There is a small, horizontal slot located just below the arrow on the back of the device, and above the back Safety Sensor. DO NOT pry, bend or pull the BLACK Safety Sensor itself, as it will break, and it has nothing to do with Scent Pouch Compartment access.
 2. Insert a new SteriShoe scent pouch. (Remove old one first if installed).
 3. Replace scent cap by pressing it back into place.
- Note: Use only SteriShoe branded scent pouches. Other scent products may void the warranty.

Indicators and trouble shooting

Indicator light						
Beeps	None	Beep (pause) beep (pause) beep (pause)	Beepbeep (pause) beepbeep (pause)	Beepbeepbeep (pause) beepbeepbeep (pause)	None	None
Situation	Operating correctly.	The ambient light sensor is detecting too much light – this is a safety feature that limits the user being exposed to the UV light. Covering up the shoe will fix this issue.	The movement switch at the bottom of the sanitizer is not depressed. This switch will deactivate the UV light if the SteriShoe is moved while treating a shoe. The switch needs to be pressed down against the bottom of the shoe to start the UV light.	The UV bulb is either not inserted correctly or it has stopped working.	Controller is in volume adjustment mode.	Fan (only) is operating.
Corrective action	None.	1. Press SteriShoe+ (Power) button to turn off. 2. Make sure the SteriShoe is completely covered within the shoe bag. Make sure the SteriShoe is gently pushed under the tongue of the shoe and is upright in shoe. 3. Press Power button to restart.	1. Press SteriShoe (Power) button to turn off. 2. Move the SteriShoe slightly forward in the shoe and gently push it down and toward the back of the shoe. There should be a subtle click. 3. Press Power button to restart.	1. Press SteriShoe (Power) button to turn off. 2. Remove the UV lamp from the side that does not have the error and switch it with the side that has the error. 3. Press Power button to restart. 4. If the Bulb error moved to the other side, a replacement lamp is necessary. If the error stayed on the same side, call customer care.	Press SteriShoe button to get out of mode. See “Adjusting the controller volume” instructions for how to change beep volume.	None. Fan is operating correctly, will operate for next eight hours or until you press the FAN or SteriShoe button.
To order replacement bulbs, scent pouches or safety bags, or if these corrective steps do not resolve the trouble, please call customer service toll free 1-800-PEDIFIX (733-4349)						

Adjusting the contoller volume

Your SteriShoe has a speaker inside the controller to inform you about safety features and other operations. To adjust the volume of the speaker:

- Press and hold the SteriShoe button
- The error LEDs will flash and you will hear a few beeps indicating you have entered the volume setting mode.
- Pressing the Left and Right button will decrease or increase the volume on your SteriShoe
- The volume level of the controller is indicated as follows:

Solid bulb light = sound off

Solid movement and bulb lights = low volume

Solid ambient, movement and bulb lights = high volume

- To save your setting and exit, press and release the SteriShoe button in the center of the controller.

See website for trouble shooting video.